



**INSTRUCTIONS FOR USE**

This leaflet is intended to inform users of Bausch + Lomb (balafilcon A) hydrophilic soft contact lenses how to wear and care for their lenses.

It applies to:

PRODUCT	MATERIAL	WATER CONTENT	PACKAGING SOLUTION
PureVision® (balafilcon A) Visibility Tinted Contact Lenses	balafilcon A	36% by weight	Borate buffered saline
PureVision® Toric (balafilcon A) Visibility Tinted Contact Lenses			
PureVision® Multi-Focal (balafilcon A) Visibility Tinted Contact Lenses			
PureVision®2 (balafilcon A) Visibility Tinted Contact Lenses	balafilcon A	36% by weight	Borate buffered saline with Poloxamine
PureVision®2 For Astigmatism (balafilcon A) Visibility Tinted Contact Lenses			
PureVision®2 For Presbyopia (balafilcon A) Visibility Tinted Contact Lenses			

**Description and wear application**

These lenses are designed for daily wear with a replacement period as prescribed by the eye care professional. They may also be used as disposable lenses to be discarded after each removal or for extended wear for 1-30 days between removal, as recommended by the eye care professional. When reusing lenses they must be cleaned, rinsed and disinfected between removal and reinsertion.

The Bausch + Lomb (balafilcon A) Contact Lenses can also be used for therapeutic use in the management of corneal epithelial disorders. Always follow your eye care practitioner's recommended wearing schedule.




These lenses are supplied sterile and are intended by the manufacturer for reuse as prescribed by the eye care professional. They have been sterilized by steam.

If the packaging is damaged or opened DO NOT USE.

**Insertion and removal instructions**

Before handling lenses, always wash and dry hands thoroughly avoiding perfumed or medicated soaps. Pour the lens into the palm of your hand. Make sure the lens is the right way out.

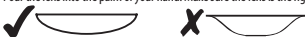

Inspect your lens before inserting and do not insert if scratched or damaged.

To remove your lens, make sure your hands are clean and dry, use your finger to slide the lens down to the white part of the eye and remove it. Use a rewetting drop if the lens is difficult to remove.

**Caring for your lenses**

- Cleaning and disinfecting your lenses between uses is essential for healthy eyes
- Use only soft contact lens care products following the manufacturer's instructions
- Always use fresh, unexpired lens care solutions
- Do not use other solutions such as tap or bottled water or saliva to rinse, store or re-wet lenses
- Always keep lenses completely immersed in the recommended solution when not being worn
- Remember to clean your lens case whenever you clean your lenses
- Always discard lenses at the end of the wear schedule prescribed by your eye care practitioner
- Always wash and rinse hands before handling lenses
- Do not allow cosmetics, lotions, soaps, creams or perfumed substances to get in the eyes or on lenses
- Never use sharp objects or finger nails to remove lenses from their containers or your eyes

**Warnings and precautions**

Failure to follow your eye care practitioner's recommended wear schedule and cleaning regime can give rise to greater risk of serious eye infections.

Regardless of the schedule prescribed for you, once a lens is removed at the end of the prescribed wearing period, your eye should have a rest period as recommended by your eye care professional.

Do not wear daily disposable lenses for longer than a day or while sleeping.

The risk of microbial keratitis has been shown to be greater among users of extended wear contact lenses than among users of daily wear contact lenses. While the great majority of patients successfully wear contact lenses, continuous wear of lenses also is reported to be associated with a higher incidence and degree of epithelial microcysts and infiltrates, and endothelial polymegathism, which require consideration of discontinuation or restriction of continuous wear. The epithelial conditions are reversible upon discontinuation of continuous wear.

Do not wear contact lenses if you are suffering from:

- Allergy, inflammation, infection or redness in or around the eye
- Dry eyes (inadequate tear fluid)
- Poor health affecting the eyes, e.g. colds and flu
- Any systemic illness affecting the eyes

Contact your eye care practitioner before wearing contact lenses when:

- Practicing water sports
- In excessively dry or dusty environments
- Using eye medication

If you experience any of the following problems, remove your lenses immediately:

- Eyes stinging, burning, itching, irritation or other eye pain
- Reduced comfort compared with previous wearing
- Unusual secretions or excessive watering from the eye
- Redness of eyes
- Severe or persistent dry eyes
- Reduced or blurred vision
- Rainbows or halos around lights
- Sensitivity to light

If any of these symptoms continue after removal and insertion of a new lens, then remove the lens and contact your eye care practitioner. These symptoms can lead to more serious conditions if ignored.

**Solution precautions**

Do not use the Allergan Ultracare Disinfecting System or any of its components (Oxysept Ultracare Disinfecting Solution, Oxysept Neutralizing Tablets, and Ultrazyme Enzymatic Cleaner) to clean and disinfect Bausch + Lomb (balafilcon A) Contact Lenses because the lens dimensions will be altered.

**Reference guide for symbols, labels and cartons**



Fee paid for waste management per 94/62/EC



Bausch & Lomb Incorporated  
1400 North Goodman Street  
Rochester NY, 14609, USA



Bausch & Lomb Incorporated  
Cork Road Industrial Estate  
Waterford, X91 V383, Ireland

Date of Issue: 2018-06



0050 Indicates the CE conformity marking and the notified body number



Temperature limitation



See Instruction for Use



Sterile using steam or dry heat



For US only: Caution Federal law restricts this device to sale by or on the order of a licensed practitioner

®/™ are trademarks of Bausch & Lomb Incorporated or its affiliates.

© Bausch & Lomb Incorporated.

Other product/brand names are trademarks of their respective owners.