

**INSTRUCTIONS FOR USE**

This leaflet is intended to inform users of Bausch + Lomb (samfilcon A) hydrophilic soft contact lenses how to wear and care for their lenses.

It applies to:

PRODUCT NAME	MATERIAL	WATER CONTENT	PACKAGING SOLUTION
Bausch + Lomb Ultra® (samfilcon A) Visibility Tinted Soft Contact Lenses	samfilcon A	46% by weight	Borate buffered saline with Poloxamine
Bausch + Lomb Ultra® for Astigmatism (samfilcon A) Visibility Tinted Soft Contact Lenses			
Bausch + Lomb Ultra® for Presbyopia (samfilcon A) Visibility Tinted Soft Contact Lenses			

Description and wear application

These lenses are designed for daily wear with a replacement period as prescribed by the eye care professional. When reusing lenses they must be cleaned, rinsed and disinfected between removal and reinsertion.




These lenses are supplied sterile and are intended by the manufacturer for reuse as prescribed by the eye care professional. They have been sterilized by steam.

If the packaging is damaged or opened DO NOT USE.

Insertion and removal instructions

Before handling lenses, always wash and dry hands thoroughly avoiding perfumed or medicated soaps. Pour the lens into the palm of your hand. Make sure the lens is the right way out.

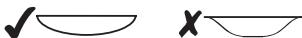

Inspect your lens before inserting and do not insert if scratched or damaged.

To remove your lens, make sure your hands are clean and dry, use your finger to slide the lens onto the white of the eye. Use a rewetting drop if the lens is difficult to remove.

Caring for your lenses

- Cleaning and disinfecting your lenses between uses is essential for healthy eyes
- Use only soft contact lens care products following the manufacturer's instructions
- Always use fresh, unexpired lens care solutions
- Do not use other solutions such as tap or bottled water or saliva to rinse, store or re-wet lenses
- Always keep lenses completely immersed in the recommended solution when not being worn
- Remember to clean your lens case whenever you clean your lenses
- Always discard lenses at the end of the wear schedule prescribed by your eye care practitioner
- Always wash and rinse hands before handling lenses
- Do not allow cosmetics, lotions, soaps, creams or perfumed substances to get in the eyes or on lenses
- Never use sharp objects or finger nails to remove lenses from their containers or your eyes

Warnings and precautions

Failure to follow your eye care practitioner's recommended wear schedule and cleaning regime can give rise to greater risk of serious eye infections.

Do not wear lenses while sleeping.

Do not wear contact lenses if you are suffering from:

- Allergy, inflammation, infection or redness in or around the eye
- Poor health affecting the eyes, e.g. colds and flu
- Any systemic illness affecting the eyes

Contact your eye care practitioner before wearing contact lenses when:

- Dry eyes (inadequate tear fluid)
- Practicing water sports
- In excessively dry or dusty environments
- Using eye medication

If you experience any of the following problems, remove your lenses immediately:

- Eyes stinging, burning, itching, irritation or other eye pain
- Reduced comfort compared with previous wearing
- Unusual secretions or excessive watering from the eye
- Redness of eyes
- Severe or persistent dry eyes
- Reduced or blurred vision
- Rainbows or halos around lights
- Sensitivity to light

If any of these symptoms continue after removal and insertion of a new lens, then remove the lens and contact your eye care practitioner. These symptoms can lead to more serious conditions if ignored.

Reference guide for symbols labels and cartons

Fee paid for waste management per 94/62/EC



Bausch & Lomb Incorporated
1400 North Goodman Street
Rochester NY 14609, USA



Bausch & Lomb Incorporated
Cork Road Industrial Estate
Waterford, X91 V383, Ireland

Date of Issue: 2018-07



Indicates the CE conformity marking and the notified body number

®/™ denote trademarks of Bausch & Lomb Incorporated. © Bausch & Lomb Incorporated.

P9WW03/ 1802321